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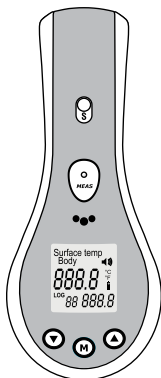
CONTENTS

General Description.....	19
1. Features.....	20
2. Safety Information.....	20
3. Recommendations for Use.....	21
4. Description.....	23
5. Display.....	24
6. Description of Symbols on the Thermometer.....	24
7. Technical Information.....	25
8. Advice for the First Use on Precision Testing.....	26
9. Procedure for Measuring Body Temperature.....	27
10. Different Measuring Functions.....	28
11. Maintenance and Cleaning.....	30
12. Troubleshooting.....	31

GENERAL DESCRIPTION

The device is designed to measure human body temperature. It is ideal for measuring body temperature of children, even during sleep. You can also measure the temperature of air, different objects, like water in the baby bath, baby food etc.

We recommend you to know your normal individual body temperature when you feel well in order to properly interpret the data that differ from the average.

Warning: In case if the body temperature is above 38 °C, we recommend you to contact your physician immediately.

I. FEATURES

- Precise non-contact measurements.
- Selectable Body and Surface temperature.
- User selectable °C or °F.
- Adjustable high temperature signal.
- Memory for 32 measurements.
- Automatic Data Hold & Auto power off.
- Automatic selection range.
- Display Resolution 0.1 °C (0.1 °F).
- Backlight LCD display.
- Colour of display changes depending on temperature level.

2. SAFETY INFORMATION

- The device must be used only for the purposes described in this user manual.
- Do not drop or knock the device, and do not use it in case of damage.
- The device must be used only in an ambient temperature range between 10 and 40 °C.
- Do not expose the thermometer to electric shocks.
- Do not expose this thermometer to extreme temperature conditions (>50 °C or <-20 °C).
- Do not use the device in relative humidity higher than 85%.
- Do not immerse the device into water or other liquids.

- Do not use the device near sources of a high electromagnetic radiation, such as cordless or cellular phones.
- Keep the device away from water and heat sources, including direct sunlight.
- Do not attempt to charge the batteries that cannot be charged.
- Keep the device in the room pending 15-30 minutes before the first use.

Warning: Before taking of the temperature make sure to remove hair, sweat, make-up or any other obstacles from the forehead. Select «**BODY**» mode to measure the body temperature and «**SURFACE**» mode to measure the surface temperature. Depending on the type and thickness of skin there can be difference in the readings of body temperature. Should a problem occur with your device please contact your retailer. Do not attempt to repair the device yourself.

3. RECOMMENDATION FOR USE

Thermometer is designed for non-contact body temperature measurement, namely for forehead and temporal region of adults and children (mode «**BODY**»). Thermometer can also be used to measure the temperature of a baby-bottle or bath, or room temperature (by using the «**SURFACE**» mode). Thermometer is not intended for rectal, oral and axillary method of temperature measurement. The factors following can influence the body temperature of a healthy person:


- Individual peculiarities of metabolism.
- Age (body temperature of infants and small children is higher and it decreases with age. Children are likely to have significant temperature fluctuations faster and more often).
- Clothes that a person is wearing.
- Temperature of air.
- Time of day (In the morning the body temperature is lower, and by the end of the day it rises).
- Preceding physical activity.
- Measurement method.
- Phase of the menstrual cycle.

Normal body temperature according to age

Age	Normal temperature, °C
0 – 2	36,4 – 38,0
3 – 10	36,1 – 37,8
11-65	35,9 – 37,6
Over 65	35,8 – 37,5


To achieve the correct and stable results during the first use of the thermometer, we advise to test it and, if necessary, adjust the device with the help of a mercury thermometer (part 8 of this manual).

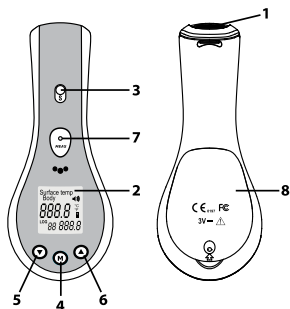4. DESCRIPTION

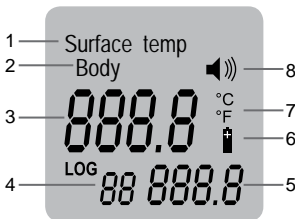1. IR Sensor.
2. LCD display.
3. Body/surface switch.
4. MODE button.
5. Button «**DOWN**».
6. Button «**UP**».
7. Measurement button.
8. Battery compartment.

5.DISPLAY

1. Surface mode.
2. Body mode.
3. Temperature display.
4. Measurement number.
5. Last measurement value.
6. Battery charge level.
7. Measuring unit (C° or F°).
8. Alarm signal value.

6. DESCRIPTION OF SYMBOLS ON THE THERMOMETER

	Supply voltage: 3V direct current
	Type of equipment: B (with higher rate of protection)
	With the aim of environment protection, please, utilize the batteries according to the set of rules

	Attention! Read the accompanying documentation
	The name and the address of the manufacturer
	Production date
	The device is in conformity with applicable requirements of MDD 93 / 42 / EEC

7. TECHNICAL INFORMATION

Parameter	Value
Body temperature range	30,0°C to 42,5°C / 86,0°F to 108,5°F
Surface temperature range	0°C to 60°C / 32°F to 140°F
Basic accuracy	±0,3°C / ±0,6°F
Measuring distance	5-15cm
Response time	0,5 s
Overrange indication	LCD will show "Hi" / "Lo"
Emissivity	0,95 fixed value
Spectral response	8~14 um
Power off	Automatically shut off after approx. 7 seconds
Operating temp.	10°C to 40°C (50°F to 104°F)
Storage temp.	-20°C to 50°C / -4°F to 122°F Humidity Rate 10%~90%RH (storage), <80%RH (operating)
Humidity Rate	10%~90%RH (storage), <80%RH (operating)
Power	DC 3V (2x1,5V "AAA" batteries)

Precision of Non-contact Measurement

From 30 to 35,9 °C	$\pm 0,3$ °C	According to standard ASTM E1965-1998 (2003)
From 36 to 39,0 °C	$\pm 0,2$ °C	
From 39 to 42,0 °C	$\pm 0,3$ °C	

8. ADVICE FOR THE FIRST USE ON PRECISION TESTING

Step 1. Take the temperature of a person using a mercury thermometer. You will get 36,6 °C for instance.

Step 2. Compare this result with the normal body temperature, depending on the measurement method as shown in the table below:

Measurement method	Normal temperature, °C
Rectal	36,3 – 37,8
Oral	36,0 – 37,5
Axillary	34,7 – 37,3
Ear	36,0 – 37,8

It should be kept in mind that the rectal method of measurement most closely reflects the central body temperature.

Step 3. Take the temperature of the same person using the non-contact thermometer keeping the 5 to 15 cm distance between the thermometer and the forehead. Remove any obstacles from the patient's forehead (hair, perspiration, make-up etc). If you get 36,6°C, the non-contact thermometer is properly set and ready for use. If you get a different temperature, 36,4°C (your difference is 0,1°C) for example, you should tune the non-contact thermometer.

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Step 4. To check, take the temperature again using the non-contact thermometer.

9. PROCEDURE FOR MEASURING BODY TEMPERATURE

- Install the batteries.
- For the first use or when replacing the batteries the device must be stored in the room at least for 15-20 minutes.
- If the device has not been used for a long time, we recommend you to measure the room temperature first.
- Make sure there is no hair, perspiration, cosmetic or cap covering the forehead. Point the device at the patient's forehead (see the picture) at a distance of 5 to 15 cm Press the trigger – the temperature will be displayed less than in 1 second.
- If there are obstacles for measuring in the forehead area or the ambient temperature is significantly different, the temperature can be measured behind the earlobe.

10. DIFFERENT MEASURING FUNCTIONS

Surface temperature mode settings

- **Temperature Unit setting — F1 menu.**
Press «**MODE**» button for 2 seconds, **F1** will appear in the display. Release the button, Press «**DOWN**» button for degree Celsius, «**UP**» button for degree Fahrenheit.
- **Alarm setting — F2 menu.**

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- **Correction Mode (Only for the Body Measurement Mode) — F3 menu.**

To adjust the total variation of the Non-contact Body Infrared Thermometer, press «**MODE**» button for 2 seconds, **F1** will appear in the display. Press «**MODE**» button a third time until **F3** appears in the display. Release the button, press «**UP**» button to increase 0,1°C (0,1°F) each time or «**DOWN**» button to reduce 0,1°C (0,1°F) each time.

- **Alarm ON/OFF setting — F4menu.**

Press «**MODE**» button for 2 seconds, **F1** will appear in the display. Press «**MODE**» button a third time, **F4** will appear in the display. Release the button, press «**UP**» button to start the buzzer and «**DOWN**» button to stop it.

- **Exiting the Setting Mode.**

Keep pressing «**MODE**» button until the display is off.

Data Storage.

To review the logged data points, press both the «**UP**» and «**DOWN**» buttons for 2 seconds while the unit is **OFF**. Use the «**UP**» or «**DOWN**» buttons to scroll through the memory locations. To clear logged data, select the «**0**» memory location and press the «**MODE**» button. The unit will beep twice to indicate memory has been cleared.

- **Replacement of batteries.**

When the LCD displays the battery sign, the battery level is low. Open the lid and change the batteries, pay attention to the correct positioning. Never use rechargeable batteries, use only batteries for single use. If the thermometer is not used for a long time, remove the batteries.

- **Longevity Use.**

The non-contact infrared thermometer was designed for professional use, its longevity is guaranteed for 40000 measurements.

II. MAINTENANCE AND CLEANING

- The Infrared Sensor is the most fragile part and must be protected carefully.
- The device can be cleaned with liquid containing 70% alcohol, do not clean the device with corrosive detergent.
- Store the device in a dry environment, and keep it away from water, dust and direct sunlight.

12. TROUBLESHOOTING

If you happen to have one of the following problems, please, refer to this user manual to help resolve the problem. If the problem persists, please, contact our customer service.

Reasons for «Lo» or «Hi» message display	Advice
Temperature reading hampered by hair, perspiration	Make sure that there is no obstruction prior to taking a temperature.
Temperature hampered by an air flux	Make sure there is no air flux as this could interfere with the infrared system
The measuring distance is too far	Please respect the measuring distance (between 5 to 15cm)
From high/low temperature condition to room temperature	Waiting for 10-20 minutes before taking the body temperature.

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