*No.1 choice  
of hospitals  
& mothers*


# Harmony™

- EN** Instructions for use
- RU** Инструкция по применению
- HU** Használati útmutató
- CS** Návod na použití
- TR** Kullanım talimatları
- PL** Instrukcja obsługi

**EN** Parts List  
**RU** Список деталей  
**HU** Alkatrészek listája  
**CS** Seznam dílů  
**TR** Parça Listesi  
**PL** Lista części

(i)

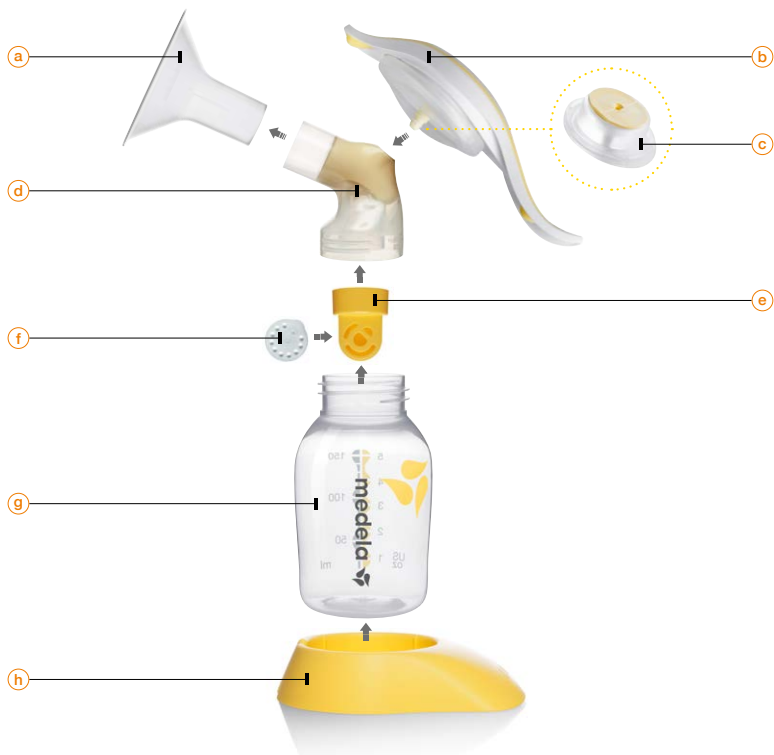**EN** Parts List  
**RU** Список деталей  
**HU** Alkatrészek listája  
**CS** Seznam dílů  
**TR** Parça Listesi  
**PL** Lista części

**a** **810.7084**  
**EN** PersonalFit™ Breastshield 24mm  
**RU** Воронка PersonalFit™  
**HU** PersonalFit™ szivófej  
**CS** Prsní nástavec PersonalFit™ (24 mm)  
**TR** PersonalFit™ Meme Hunisi 24 mm  
**PL** PersonalFit™ Lejek dzielony

**b** **800.7170**  
**EN** Handle  
**RU** Рукоятка  
**HU** rümpakár  
**CS** Madlo  
**TR** Kol  
**PL** Rączka

**c** **800.0766**  
**EN** Diaphragm/stem with O-ring  
**RU** Диафрагма/штырек с O-образным кольцом  
**HU** szivattyú membrán dugattyúval és tömítő gyűrűvel  
**CS** Těsnicí podložka madla s O-kroužkem  
**TR** Diyafram/gövde (O-halka ile)  
**PL** Membrana z trzpieniem i O-ringiem

**d** **800.7196**  
**EN** 2-Component Connector  
**RU** 2-компонентный коннектор  
**HU** csatlakozó könyökelem  
**CS** Hlava ventilu  
**TR** 2-Parça Bağlayıcı  
**PL** Konektor

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TR  
32

PL  
39

Thank you for choosing the Harmony breastpump. Breastmilk is the natural choice for your baby, but there are reasons and situations where breastfeeding is not possible. For over 50 years, it has been Medela's conviction that these mothers and babies can be supported. In close co-operation with breastfeeding experts, Medela, one of the marketleaders, has researched and developed breastpumps that perfectly meet breastfeeding mothers needs. For mothers who insist on the best for their babies and themselves. Because you care.

## Why Harmony™?

Harmony is the best solution if you need to leave your baby for short periods, for shopping or sports for example. You can still provide valuable breast milk. The 2-Phase Expression technology and the practical handling enable you to pump efficiently. Harmony is discreet and easily transported.

### What is 2-Phase Expression®?

Medela has been the first company worldwide to deeply research babies and their feeding behaviour. The results have led to the development of the 2-Phase-Expression-Technology. 2-Phase-Expression is based on the natural sucking rhythm of the baby: Short, stimulating movements (**Stimulation**) are followed by longer, more intensive ones (**Expression**), ensuring an optimal milkflow. Thus, more milk can be expressed in a shorter time – and more comfortably.

# Indications for use

Harmony is a manual breastpump for supporting breastfeeding mothers.

Harmony is ideal:

- I To help overcome mother or baby breastfeeding problems
  - To relieve sore and cracked nipples
  - To bring out flat or inverted nipples

# Important safeguards

Read all instructions before use.

- I Use the product only for its intended use as described in this instruction manual.
- I Use Medela original accessories only.
- I Check pump kit components for wear or damage.
- I This is a single user product. Use by more than one person may present a health risk.
- I If problems or pain occur, consult your lactation consultant or physician.

SAVE THIS INSTRUCTION MANUAL.

# How to clean Harmony pump set

Please follow these handling and cleaning instructions. Use drinking water quality only.  
Disassemble all parts prior to cleaning.

## Part

a b c d e f g h i

### Clean prior to first and once daily

Firstly rinse with hot water, second wash thoroughly with hot soapy water and finally rinse. Medela recommends to

✓	✓	✓	✓	✓	✓	✓	✓	✓
---	---	---	---	---	---	---	---	---

covering with cold water in a saucepan and boiling out for 5 min

✓	✓	✓	✓	✓	✓	✓	✓	✓
---	---	---	---	---	---	---	---	---

or wash in top rack of dishwasher

✓	✓	✓	✓	✓	✓	✓	✓	✓
---	---	---	---	---	---	---	---	---

or with Quick Clean Microwave Bag

✓	–	✓	✓	✓	✓	✓	✓	✓
---	---	---	---	---	---	---	---	---

or with b-well Steam Sterilizer

✓	–	✓	✓	✓	✓	✓	✓	✓
---	---	---	---	---	---	---	---	---

### Clean immediately after each use

Firstly rinse with hot water by hand, second wash with warm soapy water, finally rinse in clean hot water.

✓	✓	✓	✓	✓	✓	✓	✓	✓
---	---	---	---	---	---	---	---	---

**Note:** Food colourings may discolour components which will not affect performance. Medela assumes no liability for improper handling. You can find more information at: [www.medela.com](http://www.medela.com)

# How to assemble your Harmony pump set

Attach connector and breastshield.

Place membrane onto valve.

Push the valve onto the connector. **Note:** Ensure sidewise position of the valve.

Screw the connector onto the bottle.

Insert base of handle assembly into opening on connector until it locks into place.

## \*If the handle components have been disassembled

Place the oval shaped base of the stem into the diaphragm. Ensure that the oval shaped bases match.

Place diaphragm and stem into pump handle and press down firmly around entire edge to ensure proper seal.

# How to use your Harmony

Centre breastshield over your nipple.

Start pumping with Stimulation by pressing forward on the smaller part of the handle. Press and release quickly until your milk starts to flow.

Once your milk starts to flow, change to Expression by pressing the lower end of the handle. **Note:** the level of vacuum is controlled by compression and speed of the handle stroke.

# How to store/thaw breastmilk

Medela recommends using the following guidelines for storing and thawing breastmilk. These apply to healthy, full-term babies.

	Room temperature	Refrigerator	Refrigerator freezer compartment	Freezer
<b>Freshly expressed breastmilk</b>	Do not store at room temperature	3 - 5 days at approx. 4°C	6 months at approx. -16°C	12 months at approx. -18°C
<b>Thawed breastmilk (previously frozen)</b>	Do not store	10 hours	Do not refreeze after thawing!	

- I Never fill bottles/bags more than  $\frac{3}{4}$  full to allow space for possible expansion.
- I Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).

# Troubleshooting

Trouble	Solution
<b>If there is low or no suction</b>	<p>Check that the valve head and membrane are clean and not damaged.</p> <p>Make sure the white membrane is laying flat against the valve head.</p> <p>Ensure sideways position of valve.</p> <p>Make sure the handle components are well assembled.</p> <p>Check all connections of the kit and breastpump.</p> <p>Ensure the breastshield is well assembled and forms a complete seal around the breast.</p>

## Disposal

The unit is made of plastics and may only be disposed of in accordance with local regulations.

## ....one more question...

### Am I using the right size breastshield?

... Every mother is different and needs her personal size, therefore Medela offers five different sizes of PersonalFit breastshield. Answer some questions at <http://www.medela.com/IW/en/breastfeeding/good-to-know/right-size-of-breastshield.html> to know if the actual breastshield is the right one for you.

Visit our website at [www.medela.com](http://www.medela.com). Here you will find answers to all your questions.

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Medela AG